



Philips Avent
Nipplette

Single



SCF152/01

A simple solution for inverted nipples

Effective and easy to use

Flat or inverted nipples can be a potential problem for women wishing to breastfeed. Through gentle suction the Nipplette pulls the nipple out into a small plastic thimble-like cup.

Simple solution for inverted nipples

- Clinically proven
- Non-surgical – uses gentle suction
- Simple and discreet

When to use Nipplette

- Use with inverted or flat nipples

PHILIPS
AVENT

Specifications

Country of origin

- England

What is included

- Disposable breast pad: 2 pcs
- Niplette: 1 pcs

Development stages

- Stages: 0–6 months, Pregnancy

Design

- Discreet design

Ease of use

- Easy concealing under clothing: Push syringe to release vacuum

Easy to clean

- Clean in hot soapy water

Functions

- Correct inverted nipples: Use gentle suction

Material

- Breast pad: Dermatologically tested, Natural materials

Highlights

Non-surgical – uses gentle suction

After just a few weeks of continued wear, the nipple will stay erect. Ideally use before becoming pregnant. However, you can also use in the first six months of pregnancy to achieve long-lasting correction. Any nipple inversion, which is not congenital but has occurred recently, should be immediately checked by a doctor before the Niplette is used. Use is not recommended in the last trimester of pregnancy.

Clinically proven

The Niplette: an instrument for the non-surgical correction of inverted nipples DD McGeorge FRCS (Plast). British Journal of Plastic Surgery, December 1993.

