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PORTABLE ELECTRIC BREAST PUMP

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SACA-LEITE ELECTRICO PORTÁTIL

TIRE-LAIT ELECTRIQUE PORTABLE

TRAGBARE ELECTRIC MILCHPUMPE

DRAAGBARE ELEKTRISCHE BORSTKOLF

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INSTRUCTIONS

Please read these instructions carefully before use and keep them for future reference.

SAFETY WARNINGS

WARNING!

- This is not a medical device and therefore it cannot be used for any therapeutic and/or medical purpose.
- This product is destined for domestic use only, and must be used in accordance with the user instructions provided. Any other use is to be deemed improper and therefore dangerous. Artsana declines all responsibility for any damage caused by the inappropriate, incorrect or unreasonable use of the appliance.
- This appliance and its components must not be exposed to direct sunlight, rain and other elements. The mains adaptor is for indoor use only and should therefore not be exposed to sunlight, rain or other elements.
- This appliance should not be used by children or persons with reduced mental capabilities, or not possessing the required experience or knowledge. The product can

be used by persons with reduced physical or sensory capacity as long as under supervision or after they have received instructions for the safe use of the appliance, and have understood the hazards involved.

- Keep the appliance, its components and the mains adaptor with the power cord out of the children's reach.
 - Please ensure that children do not play with the appliance, its components or the mains adaptor.
 - In case the appliance is sold by the original owner or given to third party users, instructions for the safe use of the appliance must be handed over to them, and they must have understood the hazards involved, through reading the user manual and in particular the safety warnings contained in it, which should always accompany the appliance.
 - Always place the appliance on flat and stable surfaces which cannot be accessed by children, away from cots and beds, in particular the mains adaptor: always pay maximum care and attention! The mains cord may lead to a strangling hazard.
 - Before connecting the appliance to the mains socket, ensure that the information on the power data plate and the mains
- 

adaptor is compatible with your electricity distribution network. The data plate is placed on the mains adaptor.

- Connect the mains adaptor to a socket that is easily accessible only by an adult.
- To disconnect the appliance from the mains, remove the mains adaptor plug from the power outlet.
- Do not pull the cord of the mains adaptor or the appliance itself if connected, for disconnecting the mains adaptor from the socket.
- Please ensure that the mains adaptor is always disconnected from the socket before any maintenance service, between a use and the following one, when it is not operating or unattended.
- In order to avoid the proliferation of bacteria, it is advisable to feed the baby immediately after the expression of the milk, or to immediately store the expressed milk in the most appropriate way.
- Before each single use clean and sanitize the silicone cushion (1), the pump body (2) and the valve (9).
- The cleaning and maintenance to be performed by the user must not be carried out by children and/or people with reduced

mental capabilities or lacking experience or the required knowledge.

- Do not wet the device or the mains adaptor or its power cord or put it into water or other liquids and never place it under running water.
- If the appliance, the mains adaptor or its power cord show any sign of damage, they should be no longer used.
- Adapter power cords cannot be replaced and, therefore, in case of damage, do not continue to use the mains adapter but replace it with an identical type adapter. Attention: Do not use any mains adapter different from the one supplied with this Artsana S.p.A. product (model UE06WCP-030200SPA, Ref. 20012061000), as its use would make the product non-compliant with technical standard specifications established by the relevant EC directives affecting the safety and integrity of the product. Contact the retailer or Artsana S.p.A. to replace.
- Do not attempt to open or repair the device in case of fault or damage. Any repair must be performed only by technically qualified personnel or by Artsana S.p.A.
- These instructions are available on www.chicco.com

ATTENTION.

1. Before use, remove and dispose of all plastic bags and packaging materials and keep them out of reach of children.
2. This appliance is not a toy.
3. After removing any packaging materials, ensure that the appliance has not been damaged. In case of doubt, do not use the appliance and contact technically competent personnel or go where the product was originally purchased.
4. Do not place the appliance on delicate surfaces or surfaces that may be sensitive to water and humidity.
5. Do not perform any cleaning or maintenance operation on the breast pump while it is operating, when it is powered by alkaline batteries (not supplied) or when it is connected to the mains through its mains adaptor.
6. When using any electrical appliance, it is necessary to follow a few basic safety rules:
 - Never touch the appliance with wet or damp hands.
 - Keep the appliance and the mains adaptor away from heat sources.
 - Do not touch the appliance, its power cables and the mains adaptor, if they have accidentally fallen in water; immediately disconnect the power supply, by lowering the lever of your master switch and disconnect the plug from the mains socket before touching the appliance. Do not reuse the appliance but contact the retailer or an Artsana S.p.A. authorised service centre.
 - In case of appliance and/or mains adaptor and/or power cord failure and/or malfunction, turn them off immediately and disconnect the plug from the mains. Do not attempt to tamper with them or use them and contact a retailer or an Artsana S.p.A.'s authorised service centre.
7. The appliance must be cleaned and dried thoroughly at the end of every cycle.
8. In the case where the appliance is not to be used for a prolonged time, it should be thoroughly cleaned and the batteries (if installed) removed before storing it in a clean and dry place away from light and heat sources.
9. If you decide to discontinue the use of the appliance, remove the alkaline batteries, if present, and deactivate the mains adaptor by removing (after having disconnected the plug from the mains socket) its power cord, do not dispose of it as normal urban waste but apply waste sorting as it is a waste electrical and electronic equipment (WEEE) as requested by the current EC legislation and directives.
10. Always remove the alkaline batteries if you decide not to use the device for a long time.
11. To avoid the risk of fire or electric shock, do not step on the power cord or lay it under carpets or close to radiators, heaters or electric fires.
12. Operate the appliance within a temperature range between +15°C to +40°C.
13. Any repair/service made directly by the user may lead to severe risks to her or to any third party's safety further to irreparably damage the appliance and imply the immediate termination of the warranty.
14. The product's specifications may be subject to change without prior notice.
15. If you have any doubts on the con-

tents of this instruction manual, please contact your retailer or Artasana S.p.A..

WARNINGS

REMOVING AND INSERTING NON RECHARGEABLE BATTERIES

- Non-rechargeable batteries must only be replaced by an adult.
- To replace the non-rechargeable batteries: remove the cover, remove the exhausted non-rechargeable batteries from the compartment, insert the new non-rechargeable batteries ensuring to follow the right polarity, reposition the cover.
- Only use alkaline non-rechargeable batteries of the same or equivalent type recommended for the correct operation of this device.
- Never use different types of non-rechargeable batteries or rechargeable batteries and/or new and used non-rechargeable batteries or rechargeable batteries together.
- Never short-circuit the poles/terminals of a non-rechargeable battery or a rechargeable battery.
- Non-rechargeable or rechargeable batteries must be placed in their compartment by fitting them with the correct polarity.
- Non-rechargeable batteries must not be recharged. Do not try to recharge non-rechargeable batteries: they may explode.
- Always remove batteries if you expect not to use the device for a long time.
- Non-rechargeable/rechargeable batteries must be removed from the product prior to their recharging.
- Non-rechargeable/rechargeable batteries are to be charged under adult supervision only. Do not use rechargeable batteries for this device, as these may reduce or prevent cor-

rect operation.

- Do not employ lithium batteries, this appliance is not designed to work with lithium batteries. Using lithium batteries may create hazardous conditions.
- Always remove exhausted non-rechargeable/rechargeable batteries from the device to prevent leaks that can damage it and/or anything surrounding it.
- If the non-rechargeable or rechargeable batteries are leaking any liquid, replace them immediately and ensure that the battery compartment is clean; thoroughly wash your hands in case of any contact with the leaked liquid.
- Do not throw exhausted non-rechargeable batteries and/or rechargeable batteries into fire, disperse in the environment or dispose of them as urban waste.
- Remove the non-rechargeable/rechargeable batteries from the device before disposal at a waste sorting point.

LEGEND OF SYMBOLS:

= Equipment in compliance with the essential requirements laid down in the EC Directives in force.

= WARNING!

= Please read the instructions booklet

= Operating instructions

= Double insulation

= For indoor use only

Do not dispose of this appliance or its components as normal urban refuse. Dispose of them at an appropriate differentiated collection point, as requested by the laws in force in each country.

THE BREAST PUMP AND BREAST-FEEDING

Breastfeeding is a very important stage in a baby's and in a mother's life. For babies, latching on and sucking is not the mere satisfaction of a primary physiological need, it is also, and more importantly, a precious moment in the relationship with their mothers, full of emotions and tactile, auditory, gustatory, and olfactory sensations. Satisfying this need gives a great feeling of tranquillity and well-being to both the mother and her child.

Breast milk contains all the nutrients required to meet infants' nutritional and psychological needs at least for the first 6 months of their lives.

Chicco's electric Breast Pump helps extend breastfeeding, in the event of excess milk or when you cannot always be with your baby. In such cases you can express milk, store it, and give it to baby when required.

In order to guarantee the maximum flexibility and practicality, the portable electric Breast pump can be powered both by AA type 1.5V batteries (not supplied) and through a dedicated mains adaptor (Mod. UE06WCP-030200SPA).

LIST OF THE COMPONENTS

1. Soft silicone cover
2. Pump body

3. Motor body
4. Flow control valve
5. Control keypad
6. Battery compartment
7. Connection socket power cord plug
8. NaturalFeeling Adaptor
9. Valve
10. NaturalFeeling 0M+ 150mL Bottle
11. NaturalFeeling 0M+ teat with cover
12. Ring
13. Airtight cap
14. Base
15. Mains adapter
16. Power cable

PREPARATION

- Before using the Breast pump for the first time, please clean thoroughly all the components* as shown in the "**CLEANING**" paragraph.
- Assemble then the breast pump as follows:
- Place the valve (9) well inside on the circular duct within the base of the pump body (2).
- Push the outer edge to prevent damaging the valve and make sure that both closing edges of the assembled valve are joined together.
- Place the motor body (3) onto the pump body (2) and slide it on the guides pushing it well until reaching its end run, as shown in figure 1.
- **WARNING:** during this operation do not rotate and/or incline the motor body (3) as the guides of the pump body (2) may break.
- For better comfort, it is possible to assemble the silicone cover (1) onto the pump body (2). Insert the silicone cover well inside the cone of the pump body, and seal it around the outer rim of the pump body. Press the pads of the cover to remove any air trapped inside.
- Tighten the feeding bottle adapter of

the NaturalFeeling range (8) (included in the package) on the threaded base of the pump body (2) and then tighten the feeding bottle to the adapter.

- For feeding bottles from the Well-Being range there is not need to use the feeding bottle adapter of the Natural-Feeling range (8) as the feeding bottle can be directly tightened on the threaded base of the pump body (2).
- Use the practical base to lay the assembled product on a flat surface (14) and improve its stability.
- To use the breast pump with its mains adaptor (15), connect the plug (16) to the socket (7) of the motor body (3) and then insert the plug of the mains adaptor to a wall socket.
- To use the breast pump with alkaline batteries, open the cover of the battery compartment (6) and insert two AA type 1.5V batteries (not included) ensuring to follow their correct polarity. Close the compartment.

WARNING: always remove the batteries from the motor body (3) if the breast pump is powered through the mains adaptor (15) and/or in the case where it is not employed for a long time.

*Motor body (3) and mains adaptor (15) excluded.

USING THE BREAST PUMP

- Before each use, sanitize all the components * performing a cold sanitizing by using sterilizing liquids (following the instructions from the manufacturer), or a hot one (steam) using specific devices, or by boiling them (5 minutes), letting fully cool down all the hot sanitized components before their use.
- Wash your hands and assemble the breast pump as shown in the **PREPA-**

RATION paragraph.

- Wash your hands and cleanse your breast with water or using one of the practical Chicco breast cleansing wipes.

NB: to accommodate the milk flow, you can take a hot shower or apply a warm cloth on your breast just before expressing milk or breastfeeding your child.

- Sit down comfortably and relax.
- Fit the cup over your breast ensuring that the nipple is centred properly. You can use the breast pump both with and without the soft silicone cover (1); We recommend trying both configurations before choosing the best and most comfortable one for you.
- To switch on the breast pump, press the on/off button "a" (Fig 2) on the control keypad (5), the "a" and "c" LEDs (Fig 2) will light up.
- If the breast pump is powered by alkaline batteries, the "a" LED (Fig 2) will show their charge status:
 - **Green** the battery is full
 - **Orange** around 30 minutes of operation remaining
 - **Red** the alkaline batteries are empty
- The duration of the alkaline batteries may change based on the used brand and their charge status.
- NB: when switched on, the breast pump always starts in stimulation mode to favour milk pumping.
- After about 2 minutes of stimulation, it automatically switches to pumping at factory settings (first-time use) or at the settings saved from the previous session. During pumping, LEDs "a" and "b" (Fig 2) are on.
- It is possible to stop the stimulation stage and switch to the expression one by pressing the on/off button for a short time.

- If the stimulation must be repeated, press (with the appliance switched on) the "+" and "-" button at the same time. To switch then back to the expression stage press the on/off button for a short time.
- By operating the flow control valve (4) and the "+" and "-" buttons of the keypad (Fig 2) it is possible to adjust the expression basing on the traction level felt at the breast and/or basing on the quantity of expressed milk.
- Start by setting the flow control valve (4) on the "minimum" setting, then gradually increase by rotating the knob clockwise and/or operating the "+" button so to have a more intense expression.
- In the case where the selected expression level causes an uncomfortable sensation it is advisable to appropriately reduce it by acting on the flow control valve and on the "-" button.
- After around 8 seconds of operations without any change, the expression mode will be stored.
- You can express milk from both breasts alternatively.
- To switch the breast pump off, press the on/off button "A" for at least 2 seconds.
- **WARNING:** immediately after each single use clean carefully all the components* as shown in the "CLEANING" paragraph.
- To feed your baby with the breast milk immediately, remove the feeding bottle from the pump body by unscrewing it and then by tighten on the feeding bottle the ring with the teat (included in the package). This way, you will obtain a handy baby feeding bottle (NaturalFeeling) ready for use.
- Closing, instead, the bottle with the

airtight cap (13) (included in the package) to store the expressed milk in the fridge for up to 48 hours at 4 °C or in the freezer for up to 3 months at -18 °C.

- **WARNING:** When the breast pump is not used, always disconnect the mains adaptor from the mains.

CLEANING

- Before carrying out any cleaning operation, disconnect, if connected, the mains adaptor from the mains and the plug of the power cord from the socket (7) of the motor body.
- Separate the motor body (3) and all the components of the breast pump as follows:
 - I. Hold tightly the pump body (2) or the feeding bottle if it is still attached and the motor body.
 - II. Make the motor body shift vertically until completely extracted. No forcing is needed (Fig. 3).

WARNING: during this operation do not rotate and/or incline the motor body (3) as the guides of the pump body (2) may break.

Note: If the Chicco NaturalFeeling OM+ feeding bottle (included in the package) is used, separate the three parts (pump body (2), NaturalFeeling adaptor (8) and bottle (10) by unscrewing them.

If a Well-Being Chicco bottle (not included in the package) is used, unscrew directly the feeding bottle from the pump body (2).

- III. Remove the valve (9), located on the circular duct inside the base of the pump body (2) by gently pulling the tab on the side of the valve.

WARNING: Make sure that the valve is always removed in a gentle way; do not use any tool for cleaning it because it may be damaged, making the breast

pump impossible to use.

IV. Remove the silicone cover (1) from the cone of the pump body (2).

- Clean thoroughly all the components* using hot water and washing up liquid, rinse generously and dry carefully using a soft and clean cloth.
- Do not use abrasive products to clean the breast pump and its components.
- All the components, motor body(3), mains adaptor (15) and valve (9) excluded, are dishwasher safe provided that a delicate program is used. An aggressive type of washing may cause the material to become discoloured and dull over time.
- After every washing, store the product in a clean, dry place.

*Motor body (3) and mains adaptor (15) excluded.

PROBLEM, POSSIBLE CAUSE AND SOLUTION

Problem	Cause	Solution
The breast pump does not pump or pumps very little	Valve not assembled or not assembled well	Assemble the valve correctly
	Cup and motor body not assembly correctly	Assemble the cup and motor body correctly
	The cup does not fit well on the breast	Fit the motor body cup over your breast ensuring that the nipple is properly centred. Try with/without the soft silicone cover
	Speed and pumping intensity setting not adequate	Change the speed and pumping intensity setting to achieve a more comfortable stimulation and pumping experience by turning the flow control valve to the maximum and/or pressing the "+" button.
	Breast pump too tilted	Do not tilt the breast pump too much during use
	Exhaust and/or weak batteries	Change the batteries
Painful milk expression	Pumping speed and/or intensity too high	Change the speed and pumping intensity setting to achieve a more comfortable stimulation and pumping experience by turning the flow control valve to the minimum and/or pressing the "-" button.
The breast pump does not switch on	Failure of mains adapter and/or device	Contact Artsana S.p.A.
	Exhaust batteries	Change the batteries

If none of the above operations solves the problem, contact Artsana S.p.A.

TECHNICAL SPECIFICATIONS

Ref. 00 002328 300 000 PORTABLE
ELECTRIC BREAST PUMP

Mod. 06667 3V --- 2.0A

Noise level: below 60 dB

Type of appliance: continuous operation mode

Motor body weight: 165gr. excluding batteries

Dimensions: 98x70x105h mm

Made in Italy

Storage and transportation conditions:

Ambient temperature: $-15 \div +50$ °C

Relative humidity: $10 \div 90\%$

Operating temperature: $+15 \div +40$ °C

MAINS ADAPTER

Mod. UE06WCP- 030200SPA

INPUT: 100-240 V~ 50/60 Hz 200 mA

OUTPUT: 3.0V --- 2.0A

Made in China

Declaration of EC Compliance:

Artsana S.p.A hereby declares that this appliance, Chicco branded ELECTRIC BREAST PUMP model number 06667, complies with all the essential requirements and other provisions set forth by the EU Directives 2004/108/EC (electromagnetic compatibility) and 2006/95/EC (electrical safety) and 2011/65/EU (RoHS).

A full copy of the EC declaration can be requested from Artsana S.p.A. Via S. Catelli 1 – 22070 Grandate (Co) Italy.

This product complies with the Directive 2012/19/EU.

The crossed bin symbol on the appliance indicates that the product, at the end of its life, must be disposed of separately from domestic waste, either by taking it to a separate waste disposal site for electric and electronic appliances or by returning it to your dealer when you buy another similar appliance. The user is responsible for taking the appliance to a special waste disposal site at the end of its life. If the disused appliance is collected correctly as separate waste, it can be recycled, treated and disposed of ecologically; this avoids a negative impact on both the environment and health, and contributes towards the recycling of the product's materials. For further information regarding the waste disposal services available, contact your local waste disposal agency or the shop where you bought the appliance.

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Artsana S.p.A. - Via Saldarini Catelli 1 - 22070 Grandate (CO) Italy